

Spot-on Pouching

A user guide to the correct use of Heat Sealed Pouches

Tip

Choose the right size

Choosing the right pouch size will allow for the expansion and contraction of the pouch during the sterilisation process.



Tip

Ensure steam penetration can occur

Steam enters through the paper side of the pouch. Always make sure that the inner pouch fits into the outer pouch without folding for even steam penetration.



Tip

Labelling

Place the identification label on the plastic film, NOT on the paper side of the pouch.



Tip

Loading for Sterilisation

When loading individually, ensure that the pouch is facing upwards. This will help prevent wet packs. When loading multiple pouches, ensure they are loaded [paper to paper] & [plastic to plastic] This aids the flow of steam throughout the pouches.



Note: the melting point of the heat-seal will limit the maximum temperature at which the pack can be used. Heat-seal packaging should not be used at temperatures above or below those specified by the packaging manufacturer.



FM 591474



003

